SPECIMEN COLLECTION PROCEDURES

SPECIMEN TYPE

- Rectal swab with visible stool submitted in Cary Blair media or fresh stool in clean container provided by Precision Health Solutions.
- Specimen should be submitted to the laboratory for testing as soon as possible, but no later than 2 days after collection. If fresh stool collection, sample viability is 24 hours or less.

COLLECTION PROCEDURE

FRESH STOOL:

1. Place small amount of stool in clean container (nickel size amount).

FECAL SWAB:

1. After patient has obtained stool specimen, collect a small amount of stool by inserting the tip of the flocked swab into the sample.
2. Warning: Do not try to collect and transfer an excessive amount of fecal sample into tube. Only the swab tip only needs to be coated with sample.
3. Transfer the swab into the FecalSwab™ tube. Please confirm that maximum filling line is not exceeded.
4. Mash and mix the stool specimen against the side of the tube to evenly disperse specimen.
5. Holding near the end of the shaft, bend the swab at a 180 degrees angle to break it off at the marked breakpoint.
6. Shake the vial until the sample appears homogeneous.
7. Immediately label the sample collection container with first name, last name and date of birth.
 - *If this step is missed, your sample may be subject to rejection.*
8. Place the completed test requisition and any other supporting documentation in the pocket of the specimen bag.
9. Contact PHS to schedule courier pick-up.

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info@precision-healthsolutions.com

727-235-0886

833-288-9397