PGx Testing

SPECIMEN COLLECTION PROCEDURES

SPECIMEN TYPE

- Buccal swab (saliva swab) provided by Precision Health Solutions.
- Specimen should be submitted to the laboratory for testing as soon as possible. Prior to shipping, swabs should be stored in a refrigerator.

COLLECTION PROCEDURE

The donor must not eat, drink (apart from water), brush teeth or chew gum one hour before sample collection.

1. During sample collection when handling the swab applicator, the collector must hold the swab from the handle or the upper part of the swab shaft.
 - Please do not touch the swab.
2. While holding the swab from the handle or from the upper part of the shaft, remove it from the tube.
3. Holding the swab by the handle, insert the swab inside the mouth over the tongue and wet the swab with saliva.
4. Move the swab to the side of the mouth and firmly roll the swab 10 times against the cheek, close to the gingival wall.
 - Use the index and medium finger to press on the cheek while collecting the swab.
5. Repeat step 3 by transferring the swab on the other cheek.
6. Replace the swab into the plastic tube and close tightly.
7. Label the specimen container with the patient name, date of birth, and collection date.
 - If this step is missed, your sample may be subject to rejection by the lab.
8. Place closed collection device, completed Patient Requisition Form, completed Medical Necessity form, and a copy of the patient's insurance information back into the Biohazard transport envelope.
9. Prior to shipping, swabs should be stored in a refrigerator.

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info@precision-healthsolutions.com

727-235-0886

833-288-9397