UTI Panel

SPECIMEN COLLECTION PROCEDURES

SPECIMEN TYPE

- Preferred sample: Clean catch urine in sterile container or in urine collection kit provided by Precision Health Solutions.
- Sample stability is 24 hours if urine is submitted in sterile container (yellow tube or urine cup from urine kit) and 48 hours if submitted in the boric acid stabilization tube (gray tube from urine kit).

COLLECTION PROCEDURE

1. Wash hands with soap and water.
2. Use gender specific instructions:
 - **Female**
 - Spread labia apart with one hand and wipe with supplied Castille soap towelette from front to back. Repeat this process with a second fresh towelette.
 - While holding the labia apart, begin to urinate into toilet for a few seconds. Hold the urine flow while moving the urine container into the urine stream. Reinitiate urine flow into the container, allow the container to collect approximately 10 ml of urine (or half full).
 - **Male**
 - Wash the head of the penis with Castille soap towelette. If uncircumcised, retract the foreskin and hold while cleansing the head of the penis.
 - Initiate urination into toilet for a few seconds. Hold the urine flow while moving the urine container into the urine stream. Reinitiate urine flow into the container, allow the container to collect approximately 10 ml of urine (or half full).
3. Finish urinating into the toilet if required.
4. Screw the lid onto the container and return to the nurse.
5. Remove the yellow seal from the blue urine container.
 - Please be careful as under the seal contains a sharp.
6. Take grey tube and place the top of sharp container and the urine should begin filling up the tube.
 - Please repeat the same process with the yellow tube.
7. Immediately label the sample collection container with first name, last name and date of birth.
 - *If this step is missed, your sample may be subject to rejection.*
8. Place the completed test requisition and any other supporting documentation in the pocket of the specimen bag.
9. Contact PHS to schedule courier pick-up.

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