Wound Panel

SPECIMEN COLLECTION PROCEDURES

SPECIMEN TYPE

Tissue, swab of skin or wound of suspected infection site shipped in Viral Transport Media (VTM) provided by Precision Health Solutions.

COLLECTION PROCEDURE

1. Using sterile saline/water, cleanse the wound edges to remove contaminating flora.
2. Remove contaminating material such as tissue slough, necrotic tissue, dried exudate and adhered dressing residue.
3. Use care to ensure that the swab only comes in contact with wound surface.
 - Do not touch the swab below the score line. Only handle the swab from the top.
4. Use the appropriate process:
 - **Tissue Sample:**
 - Tissue sample placed in VTM.
 - **Superficial Wound:**
 - Place the swab onto the wound bed being cautious to avoid the wound edges.
 - While pressing pressure against the tissue, rotate the swab 2-3 times collecting exudate if present.
 - Using a Z-track method, run the swab over the wound bed to collect samples from the entire wound bed while continuing to rotate the swab.
 - **Wound with Tract:**
 - Cautiously place the swab into the wound tract to the deepest point if possible.
 - Once at the deepest location within the wound, rotate the swab 2-3 times while firmly pressing against the side of the wound tract.
 - Remove the swab while continuing to rotate.
5. Insert the swab into the supplied VTM tube and snap off the excess swab at the scored break point.
 - Be careful not to contaminate the lower portion of the swab. If contamination is suspected, please recollect.
6. Secure the cap onto the tube.
7. Immediately label the sample collection container with first name, last name and date of birth.
 - If submitting multiple samples, please label specimen bag with sample source.
 - *If this step is missed, your sample may be subject to rejection.*
8. Place the completed test requisition and any other supporting documentation in the pocket of the specimen bag.
9. Contact PHS to schedule courier pick-up.

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info@precision-healthsolutions.com

727-235-0886

833-288-9397